

ENDOCART™

Transport Trolley



Marketed & Promoted by

Innovative Therapeutics Pvt Ltd

8/467/2A1, Nehru Street, 2nd Cross Road, 4th Lane, Arthi Nagar,
Saraswathi Nagar, Thirumullaivaoyal, Chennai, Tamil Nadu 600062

+91 44 3559 4099 | +91 93811 27879

sales@itplmedical.com | www.itplmedical.com

Manufactured by

Overview

- Transport Trolley protects your endoscopes and other delicate equipment by minimizing direct handling and reducing contact with airborne contaminants during transport and short-term storage.
- Protected short term storage of clean scopes.
- Helps prevent damage to equipment.



- Its patented design creates a protective environment for each disinfected endoscope. Green (clean) and red (contaminated) color-coded liner covers provide immediate identification of clean or contaminated endoscopes.
- Clear identification of clean and contaminated.
- Reduces potential for cross-contamination

From Endoscope Washer Disinfector and back in 4 easy steps

1. Line the tray using the sterile base liner ensuring the inside surfaces and rim are fully covered
2. Place the clean endoscope inside the tray with the sterile red liner for later use.
3. Cover the top of the tray with the sterile green liner which is marked 'CLEAN' to allow easy identification that the scope is ready for patient use.
4. Post procedure, flush & wipe the scope and place inside the tray. Cover the tray using the red liner which clearly indicates the equipment is 'CONTAMINATED'

